

## INSTRUCTION FOR USE

### INSTRUCTION MANUAL

Katalog No: A00361

Components: Caries determining agent, water, glycerin, blue-violet (food-cosmatic) dye and auxiliary substances.

Packaging: 3 gr. syringe

### PROMARKER C - Caries Identifier Gel

#### 1-Description

PROMARKER C is used for the detection of caries in dentin.

#### 2- Usage area / Indications

Caries indicators, such as PROMARKER C, are materials used during filling to help physicians distinguish between discolored dentin and carious dentin. Although discolored dentin is not contaminated with bacteria and is softer than healthy dentin, it should not be removed, whereas infected dentin must be removed before filling.

#### 3- Usage

- Fully isolate the thread
- After washing and drying the decayed tooth, apply one drop of PROMARKER C to the affected area.
- Wait 10 seconds and rinse with water.
- Remove the blue-purple colored outer decay area from the tooth
- Repeat this procedure as many times as necessary until no more areas are blue-violet stained, i.e. the carious dentin has been completely removed.
- After the blue-violet stained tissues have been removed, the cavity is ready to be filled.

#### 4- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

#### 5- Side effects

Avoid direct contact with skin, eyes or clothing. In case of eye contact: rinse thoroughly with water and consult a doctor. In case of contact with skin or clothing: wash immediately with soap and water.

#### 6- Warning:

To avoid cross-contamination or infection, do not apply the product directly from the bottle or syringe into the caries cavity (use a disposable application tip). Avoid direct contact with skin, eyes or clothing. In case of eye contact: rinse thoroughly with water and consult a doctor. In case of contact with skin or clothing: wash immediately with soap and water. Apply the product only to the tooth being treated. Make sure that the product does not come into contact with other existing fillings, especially in marginal areas. Discoloration in these areas is very difficult to remove.

**7- Storage conditions:** Store at room temperature between 5°C – 30°C as recommended. Protect from moisture and extreme heat and cold.

#### NOTE:

Our products are produced for use in dentistry. It is not suitable for use other than professional dentists. Since the use of the products is under the control of the user other than us, the user is completely responsible for the applications. We recommend using it in accordance with the information and warnings given in the instructions.

**\*\* Disposal:** Products should be treated according to their biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.



PROMİDA DIŞ MALZEMELERİ SAN. TİC. LTD. ŞTİ.

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR

[info@promida.com.tr](mailto:info@promida.com.tr) [www.promida.com.tr](http://www.promida.com.tr)